
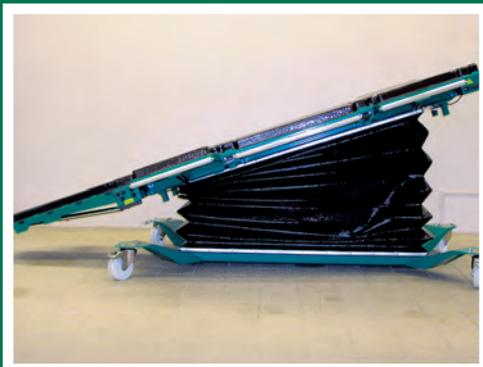


Haico PRO3

- Boening Edition -

New masterpiece for veterinary professionals!

4-WAY
TILT!

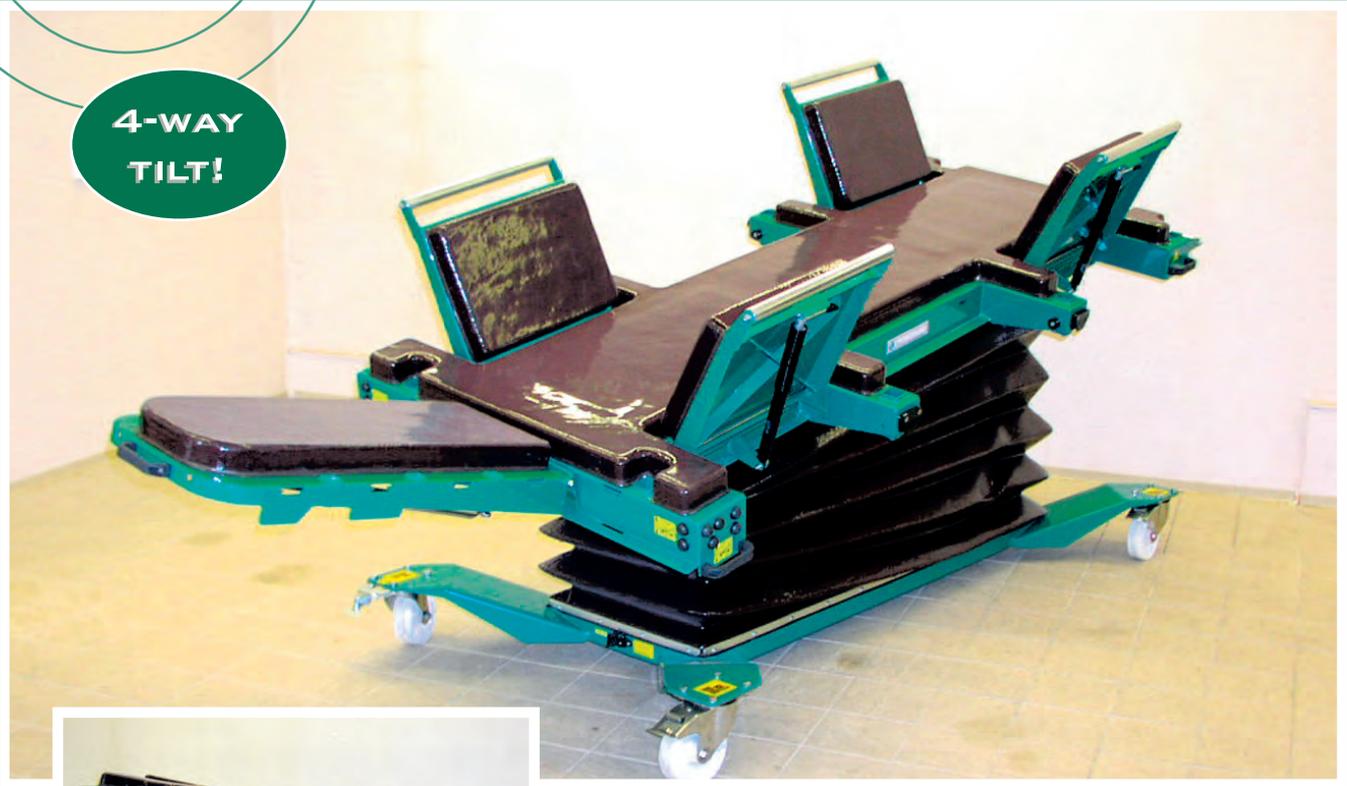

Main benefits of the new Haico PRO3 table:

- 4-way tilt **NEW!**
- longer and wider table top
- longer removable middle panel
- all side panels are easily removable
- head support can be moved sideways
- control buttons located in front
- two speeds on lifting
- new special treatment on all metal surfaces
- special battery

Don't compromise on
Quality

 **haico**
www.haico.fi

Your local distributor:

Main benefits for the customer of the new Haico PRO3 *-Boening Edition-* table:

1. 4-way tilt, longitudinal +/-25 deg (pelvic angle required for laparoscopic surgery called Trendelenburg position) and lateral +/-12 deg.

There are some latest indications in abdominal surgery where a lateral tilt is helpful.

2. Table top is 20 cm longer and 5 cm wider than in Telgte II.

Telgte II table is mainly designed for horses like medium size warmbloods and thoroughbred race horses. This new model table top is also suitable for bigger warmbloods and any other breed that are bigger. This gives more possibility to position the horse on the table. However it can be suited for smaller horses perfectly as well.

3. Removable middle panel is 15 cm longer than in model Telgte II.

This will allow much more mobility to the surgeon spec. in abdominal surgery.

4. All side panels are easily removable. Head support can be moved sideways from end to end. Head support sides are removable.

All this will allow perfect access of the surgeon and nurses in what ever position the horse is on the table. Also horse can be positioned ideally for any specific need.

5. Control buttons located in front.

Easy access and location. This is where during induction of general anesthesia always nurses are busy. Wireless remote control also included for fluent operation of the table top.

6. Two speeds on lifting: normal speed when horse is on and quicker speed for lifting unloaded table.

Double speed (from bottom to up 10 sec.) on lifting will make the operation of the table when empty faster and more convenient. Nurses do not have to stay on their knees for a long time moving the table from down position to up, this especially when the surgery is over and they are cleaning the table. Also saves battery load. This character was hoped by many nurses.

Normal speed is adapted when the patient is on the table.

7. New special treatment on all metal surfaces.

Will protect from corrosion for several years ahead.

8. Special battery.

10 years service life plus many smaller technical improvements.

9. Purpose designed extra air padding unit (option).

Allows easy access from all directions to the horse. Supports on important areas where pelvis and wrist are resting in dorsal recumbence.